HUMANTECHNIK

cm-light

D

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Kommunikationssystem

Seite 2

GB

Operating Instructions

communication system

Page 16

F

Mode d'emploi

Système de communication

Page 30

NL

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Communicatiesysteem

Pagina 44

I

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Pagina 58

E

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Página 72

Allgemeine Informationen

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General information

The CM-light communication system makes it noticeably easier for hearing-impaired people to take part in conversations in difficult situations.

Each time you switch it on, take care to ensure that the volume on the CM-light receiver is set correctly to suit your personal needs in order to avoid possible damage resulting from excessive volume. To do this, turn the volume control down low.

The CM-light communication system contains small parts which can be swallowed and is not suitable for children under the age of 3.

If the device is not used properly, the manufacturer will not accept any liability for damage and injury.

General information	page 16
Standard scope of delivery	18
Commissioning and description of the controls	
The CM-light communication system	19
Transport pouch	22
CM-light charger	23
The CM-light communication system in everyday use	
At a presentation	24
Meeting around a table	25
In direct dialogue	26
On a trip – in a car, in a train compartment or on an airplane	26
Changing the sound pattern	27
Appendix	
Safety instructions	27
Environmental advice/Disposal	28
Maintenance and care	28
Warranty	28
CE symbol	29
Technical data at a glance	29

Standard scope of delivery


The complete pack contains the following components:

No.	Device component	Quantity delivered
1	CM-light communication system	1
2	Headset	1
3	CM charger	1
4	Power supply unit	1
5	Carrying loop	1
6	Pin to set the sound	1
7	Transport pouch	1
8	Battery ("AAA")	2
	Operating instructions	1
	Warranty card	1

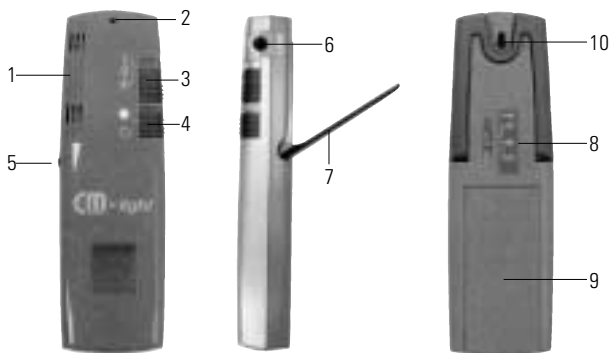Commissioning and description of the controls

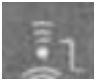
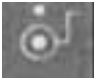
The CM-light communication system

No. Control feature Function and application CM-light


1	Microphone unit	Two microphones a defined distance from the area where the sound is picked up.	
2	LED (red)	Battery charge status when device is switched on	
	●	Continuous light	Battery is adequately charged
	◐	Flashing light	Will need to be recharged soon, approx. 2 hours "reserve"
	○	No light	Battery discharged. Place the CM-light on the CM charger to recharge the battery.

No. Control feature Function and application CM-light

3	Mode selector switch	Switch setting	Super Zoom		Dialogue: audibly reduces disruptive sounds from the sides and from behind. When the device is switched on, the change to this switch position is indicated acoustically by three signal pulses.
			Zoom		Meeting: audibly reduces disruptive sounds from behind and, at the same time, amplifies signals from in front. When the device is switched on, the change to this switch position is indicated acoustically by two signal pulses.
			Omni		Presentation: amplifies the signals from all directions. When the device is switched on, the change to this switch position is indicated acoustically by a signal pulse.
4	On/off switch	Switch setting	On ●	Push the switch up to activate your CM-light	
			Off ○	Push the switch down to activate your CM-light	
5	Volume controller	Switch setting	0	No effective sound transmission	
			1,2	Low volume. (Recommended setting during commissioning)	
			3,4	Medium volume	
			5,6	High volume	
6	Headset jack	Option for connecting a headset directly			
7	Stand	Fould-out stand			

No. Control feature Function and application CM-light

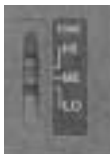- 8** Sound pattern selector switch There is a slide switch on the back of the device for setting the following 3 frequency ranges:

Switch setting	HI	"High" sound control for more treble
	ME	"Medium" sound control for equal treble and bass
	LO	"Low" sound control for more bass

Slide the selector switch to the required position using the supplied pin.

- 9** Battery compartment cover The battery compartment is located on the back of the device and holds both batteries.
-

- 10** Recording device pendant cord Insert the knob of the pendant cord into the circular part of the device and engage the knob perceptibly by pushing it towards the tip of the device.
-

Inserting the batteries into the CM-1 transmitter:

The device runs on two rechargeable "AAA"-type batteries. It can also run on non-rechargeable batteries.

Where necessary, only use original replacement batteries.

If you run your CM-1 communication system on disposable batteries, don't place the transmitter onto the CM charger.

Ensure that the batteries are placed the right way around into the battery compartment. The stipulated installation direction is indicated on the bottom of the battery compartment.

N.B. This battery is not adequately charged upon delivery and has to be charged fully on the CM charger first of all.

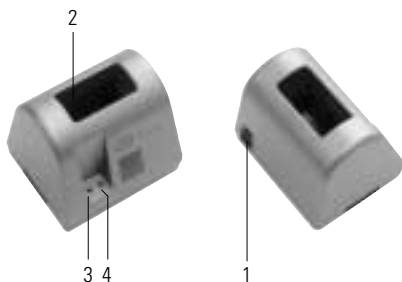
1. Switch off the CM-light transmitter.
2. Remove the battery compartment cover
3. Place the supplied batteries into the battery compartment (note the polarity!).
4. Place the cover onto the battery compartment.
5. Place the CM-light receiver into one of the CM chargers.

The green LED on the CM charger flashes throughout the charging process.
The battery is fully charged after 12 hours.

Transport pouch

CM charger


No.	Control feature	Function and application	CM-charger
1	12 V Voltage unit	Jack for the power supply unit on the back of the CM charger.	
2	Charging cradle for device components	3-pin charging cradle to hold the CM-light unit	
3	LED (red)	Standby indicator	
4	LED (green)	Charge indicator	

Installation of the CM charger

1. Plug the power supply unit into the 230 V mains.
2. Connect the CM charger to the power supply unit by inserting the round connector of the power supply unit into the DC jack on the back of the CM charger.

The CM-light communication system in everyday use

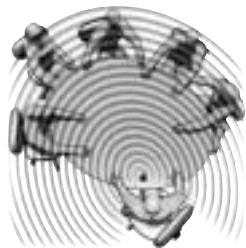
At a presentation

- You are seated in groups around a number of tables in a presentation room, for example.
- Sound is distributed evenly throughout the room via a loudspeaker system.
- Occasional questions are put to the speaker from people seated at the groups of tables.
- Use **"Omni"** transmission mode.

1. Set your hearing aid to "T" or "MT" in order to transmit sound information directly to the T-coil in your hearing aid via the teleloop, or connect a headset to a CM-light.
2. Slide the mode selector switch to the Omni position.
3. Position your CM-light as centrally as possible, e.g. on the conference table.
4. Use the fold-out stand to raise the microphone unit somewhat and to mechanically dampen jolting noises from the surface of the table.
5. Switch the CM-light on at the on/off switch.
6. Adjust the transmission volume to a comfortable level by setting the volume controller on your CM-light to an appropriate level.

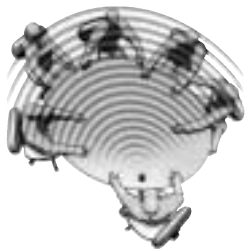
Meeting »around« a table

- You are taking part in the meeting shown.
- The other people are speaking at different levels.
- Use the "Zoom" transmission mode to reduce disruptive sounds from behind perceptibly and, at the same time, to amplify speech in front of you.

1. Plug the headset into your CM-light.
2. Slide the mode selector switch to the Zoom position.
3. Position your CM-light as close as possible to the speaker, whereby the device's microphone unit should be pointing as directly as possible towards the speaker.
4. Switch the CM-light receiver on at the on/off switch.
5. Adjust the speech volume to a comfortable level by setting the volume controller on your CM-light to an appropriate level.

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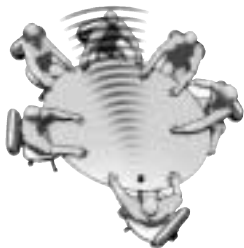- You are having a more private conversation with another person.
- You would like to concentrate fully on this person.
- You are distracted by all the other types of noise.
- Use "Super Zoom" transmission mode to reduce all noises from the sides and from behind perceptibly and to filter out the other person.

1. Change the position of your CM-light so that it is pointing as directly as possible towards the other person.
2. Slide the mode selector switch to the Super Zoom position.
3. Adjust the transmission volume to a level which is reasonable for the conversation.

On a trip – in a car, in a train compartment or on an airplane

You are on a journey, for instance as part of a travel group on a train or bus, accompanied by someone in the car or traveling alone by air. You want to be able to chat to the person sitting beside you during the trip, however the person isn't sitting on the same side as your hearing aid.

The situation is compounded by all the other disruptive sounds on board, especially the permanent noise from the air-conditioning, in an airplane for example.

Just like having a private conversation at a meeting, you need to amplify all the sounds made by the other person with the greatest possible concentration.

The CM-light produces the desired effect in Super Zoom transmission mode by reducing all noises from the sides and from behind perceptibly and by filtering out the conversation with the other person.

Changing the sound pattern

The CM-light possesses three different sound pattern ranges.
The different ranges are called "LO", "ME" and "HI".

1. Turn the CM-light around so that it is facing away from you.
2. Use the supplied pin to set the sound pattern selector switch on the back of the CM-light to the required range.

Appendix

Safety instructions

The CM-1 communication system is designed in accordance with the current state of the art and recognized safety rules. Pay attention to the following safety instructions:

- Position the charger on a non-slip base.
- Lay the connecting cable so that it cannot be tripped over.
- Never expose the device to the following sources of influence:
 - heaters
 - direct sunlight
 - moisture
- Protect the device from damp, dust, aggressive fluids and vapors.
- Only connect the connecting cables to the designated sockets / jacks.
- Only connect approved accessories.
- Under no circumstances may modifications of a technical or non-technical nature be made to the communication system.
- The CM-1 communication system contains small parts which can be swallowed and is not suitable for children under the age of 3.

Disposal

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Maintenance and care

The *CM-light* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

Warranty

The warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorized by Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer as well as a copy of the sales slip. Always specify the product number in any event.

CE symbol

The *CM-light* conforms to the requirements of EU Directive 1999/5/EC on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity.

Conformity with the afore-mentioned directive is confirmed by the CE symbol on the telephone.

Technical data

CM-light

Dimensions (H x W x D):	19 x 37 x 120 mm
Sound pressure level with used with a headset:	max. 118 dB
Weight:	64 g
Power supply:	2x 1.2V NiMH batteries (LR03 type)
Charge time (following complete discharge):	max. 12-14hrs
Operating time (load-dependent):	max. 8-10hrs

CM charger

Dimensions (H x W x D):	67 x 83 x 65 mm
Weight:	94 g
Power supply via a PSU:	Primary 220/230 V 50-60 Hz AC / Secondary 12 V DC

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Humantechnik GHL AG