HUMANTECHNIK

*InfraLight* LR

**D**

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Seite 2

**GB**

**Operating Instructions**

Page 12

**F**

**Mode d'emploi**

Page 22

**NL**

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Pagina 32

**I**

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Pagina 42


**E**

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Página 52

1
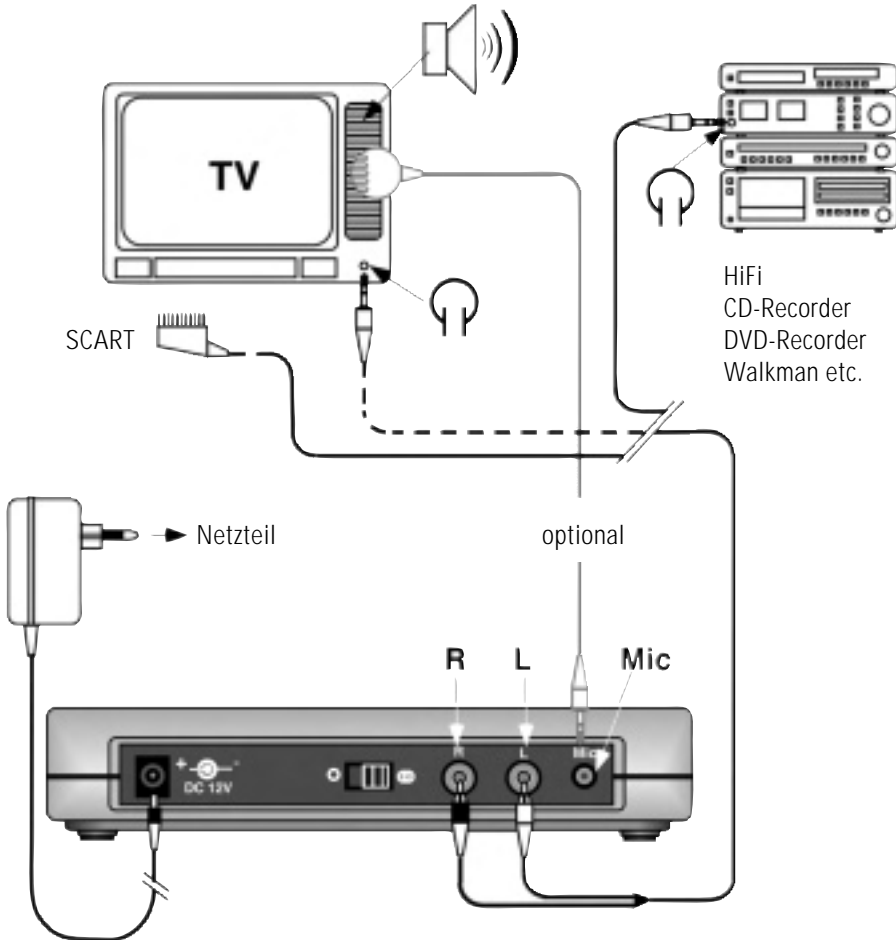
## Anschluss des Senders

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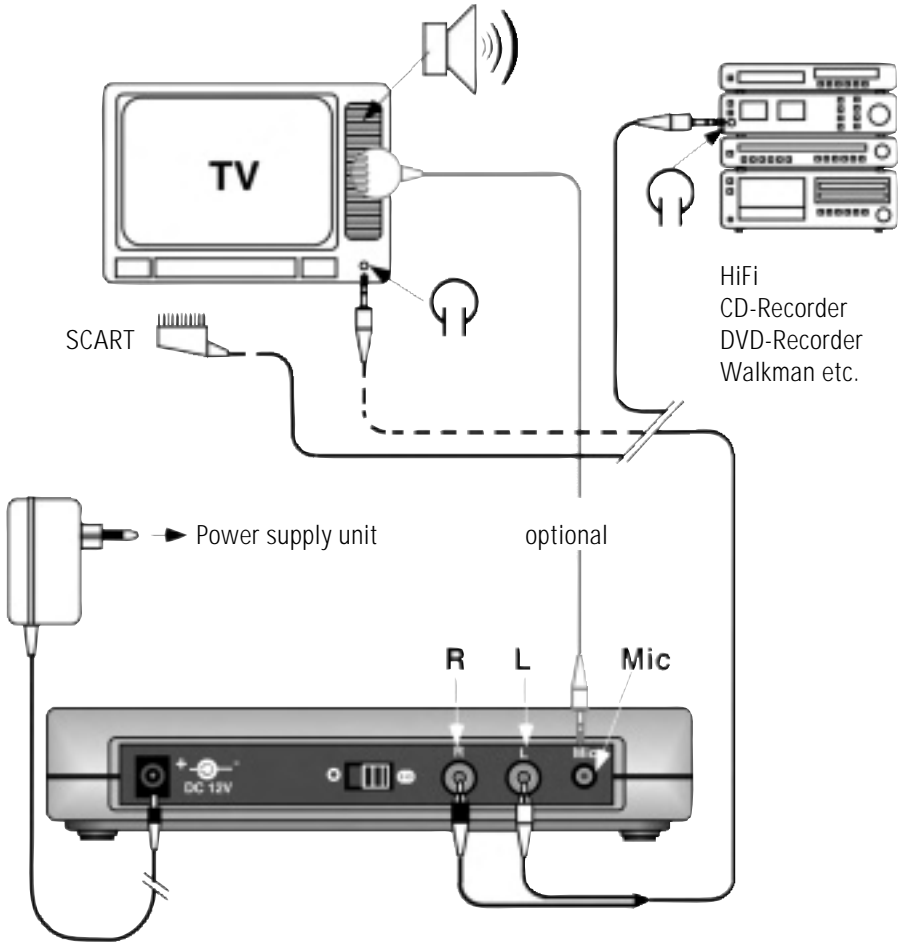
# 1 Assembling the transmitter


The volume adjustment of the TV, HiFi, CD should be set to a comfortable listening level for other members of the household.

2

## How it works

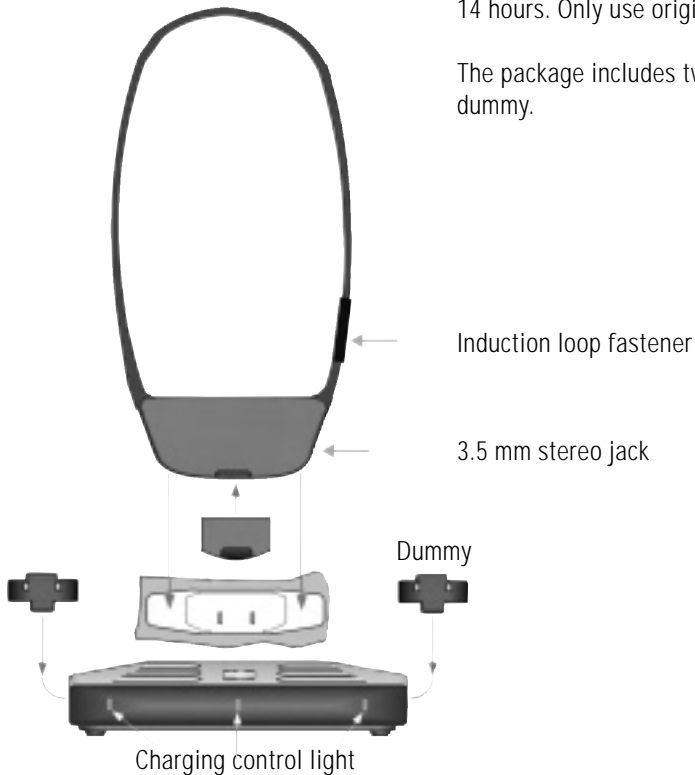When you first charge the battery packs for the headset receiver let them charge for at least 14 hours. Only use original battery packs.

The package includes two battery packs and one dummy.


The induction loop fastener makes it easy to put on the device and minimizes the risk of accidents.

Use the ON/OFF switch on the back of the receiver to switch it off and on.


**Important:** Always make sure to switch the receiver off when you've finished using it. Otherwise, the battery will run down.

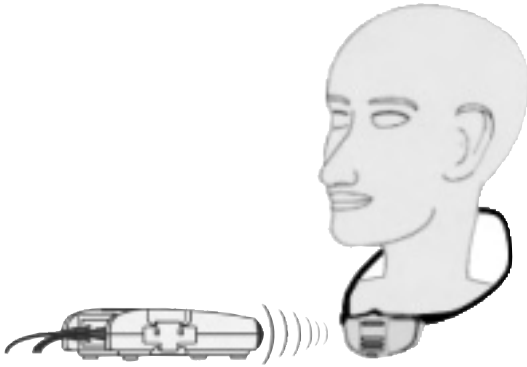You will recognize the dummy because it has no metal contact surfaces.


Please make sure that the »line of sight« between your transmitter and receiver is not obstructed.

### 3 Operating Range

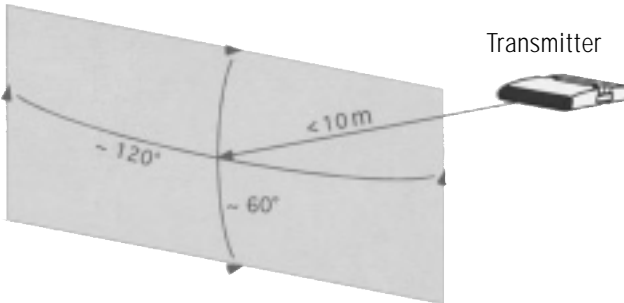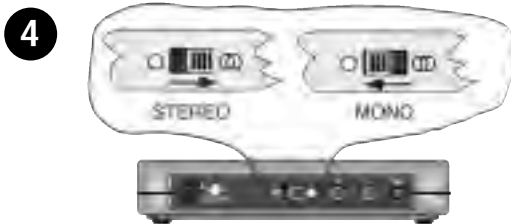The diagram below indicates the operating range. Reception will be cut off if the receiver is outside this area, approx. 10 m max. from transmitter.

### Mono / Stereo Switch

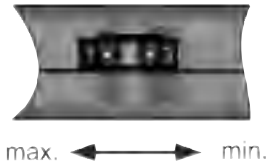
**Note!** If the transmitter is set to »Stereo« but connected to an audio source via a mono plug, sound will only be heard in one ear piece. If this occurs move the switch to the »Mono« position.

**5**

### Using the teleloop

Switch the receiver on and set your hearing aid to inductive reception. Now you can adjust the volume by turning the knob to any position between 1 and 5 (Fig.: Volume control).

- 1 means lowest volume
- 5 means highest volume

Volume control

**6**

### Using the 3.5 mm stereo output jack

The following accessories can be connected to the 3.5 mm stereo jack:

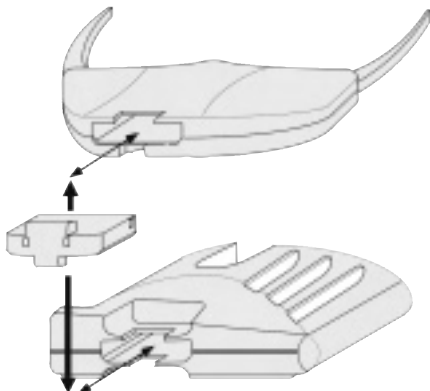
- Under-the-chin receiver, open system
- Under-the-chin receiver, closed system
- Earphone, inconspicuous design
- Lightweight headphones, conventional design
- Induction plate: monaural
- Induction plate: binaural
- Monaural Euro cable, damped
- Binaural Euro cable, damped

As soon as you no longer need to use the product, make sure to unplug any accessory you may have connected to the 3.5 mm stereo jack and flip the switch on the back of the receiver to "OFF".

## 7 Battery pack replace

If the sound quality of the receiver becomes noisy and distorted you should replace the battery pack with a charged one or charge the receiver, including battery, in its cradle.

**Warning: do not force the battery pack into the charger or receiver.**


### How to get the most out of your rechargeable batteries

Please make sure that both batteries are used on a regular basis. Constantly recharging the batteries will shorten their service life. That's why you should exchange the batteries in the hearing aid with the batteries in the charging station regularly (for best results, once a week).

## 8 Replacing the silicone covers with the transparent cover of the volume control:

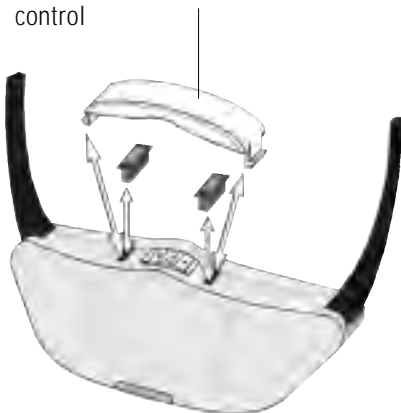Transparent cover for the volume control

The transparent cover has the following protective functions:




- a) It prevents any unintentional changes in the volume level.
- b) It protects dust and liquids from coming into contact with the volume control.

## 9 Receiver: Trimmer adjustments - Balance:

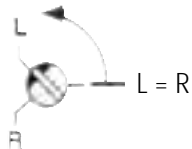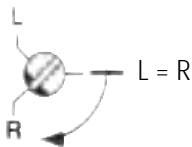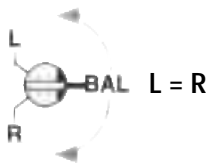
Equal on left  
and right

left - 40 dB

right - 40 dB

## 10 Please note the following informations:

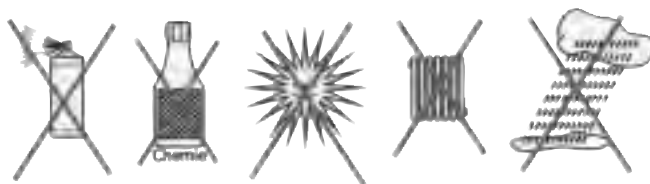To prevent the receiver from getting dirty, use a hygiene cloth from our Hygiene Set (A-4982-0) or a soft, clean, slightly damp cloth to clean it occasionally. Any product damage caused by improper treatment of the device or malfunctions due to excessive exposure the dust/dirt will render the product guarantee null and void.

### Important Notice

This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Symptom	Cause	Solution
Recharging control lights. Don't light up when batteries are placed into the battery compartments properly.	Transmitter not connected to the power adapter or no power supply.	Make sure that the power adaptor is connected to the mains power socket and to the transmitter.
Recharging control light does not light up on the same side of the transmitter where a battery is inserted.	No contact has been made between the battery and the recharging contacts of the compartment of the transmitter.	The battery has not been correctly and completely inserted into the battery compartment.
The middle recharger control light does not light up despite having the receiver placed into its compartment in the transmitter.	No contact has been made between the battery in the receiver and the recharging contacts of the compartment in the transmitter.	The receiver has not been properly placed into the compartment. Make sure it is facing in the correct direction.
	There is no battery inserted into the receiver.	Insert a battery into the receiver.
Receiver doesn't work. No audio signals being received.	The transmitter is not correctly connected to the TV or HI-FI audio output.	Please check and make sure the audio connection cables to the TV or HI-FI are properly and correctly connected. Please read through the owner's manual for your TV or HI-FI equipment to make sure if the output for headphone or line output is connected and it does work correctly.
	Your TV or HI-FI equipment is not turned on.	

<b>Symptom</b>	<b>Cause</b>	<b>Solution</b>
Receiver doesn't work. No audio signals being received.	The receiver is not switched on.	Use the ON/OFF switch on the back of the receiver to switch the device on.
	No battery is inserted into the receiver.	Insert a battery into the receiver.
	Battery is discharged	Replace battery with a fully charged one.
The audio transmission is of poor quality, noisy, distorted, unclear or no audio signals being received.	The "line of sight" between your transmitter and receiver is obstructed or you are too far away with your receiver from the transmitter.	Make sure that the front side of the transmitter is not obstructed and that it is a free "line of sight" transmission possibility. Make sure you are not located too far away from your transmitter.
	Battery is discharged.	Replace battery with a fully charged one.
	The system as used with a Plasma TV.	Plasma TV's disturb the infrared transmission. Please use one of our FM TV listening system.
Battery pack no longer gives the expected operating time per charge.	Battery pack has reached its recharging limit.	Replace battery pack with a new one.

## Warranty

This product is very safe to operate and provides highly reliable service. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will not only be carried out under warranty if the complete warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

## Disposal

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units.

You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

## Technical data

Modulation frequency	FM, Stereo and Mono
Carrier frequency	2,3 / 2,8 MHz
Frequency response	15 - 20000 Hz
Distortion	< 1 %
Noise level	typ. 60 dB
Battery charging time	approx. 14 h

### Receiver LR:

Operating time per charge	approx. 6-7 h
Maximum volume SPL	approx. 120 dBA
Weight	56 g

### Transmitter:

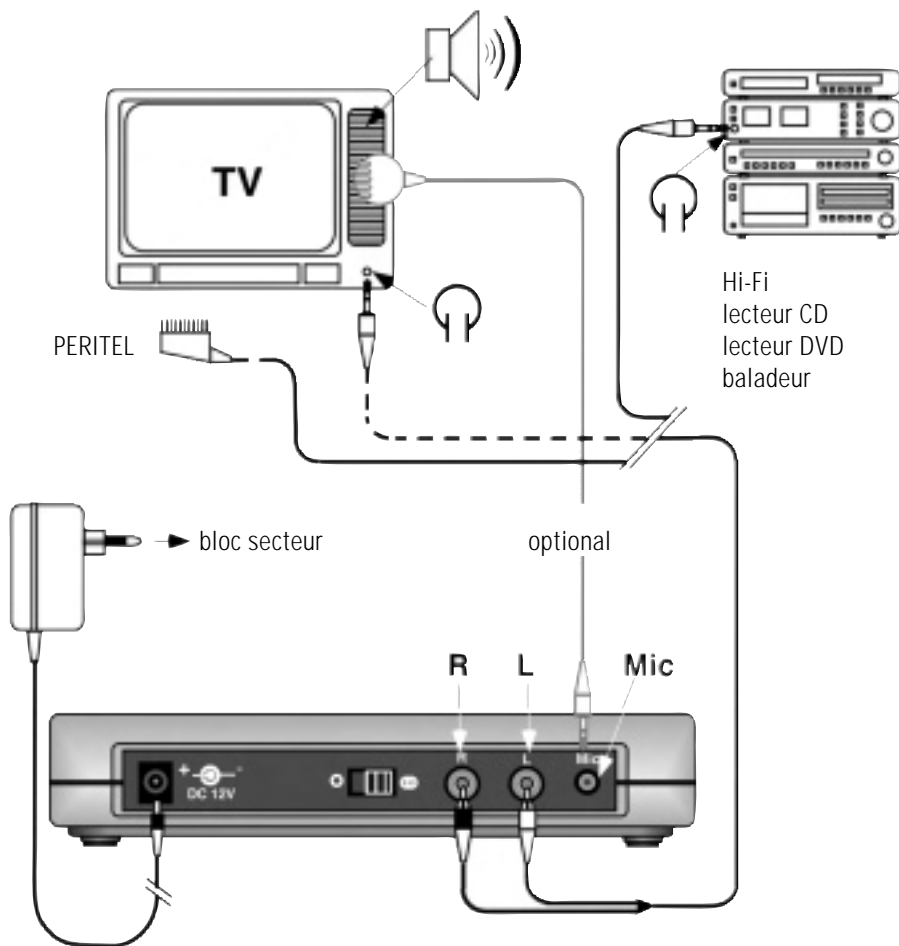
Power consumption	approx. 3.6 VA
Operating voltage	12 Volt DC
Supply voltage	230 Volt 50 Hz
Weight	approx. 172 g

This product complies with European Union norms.  
You can obtain EC compliance declarations from your dealer,  
or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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### **Humantechnik GHL AG**