# ***LPU-1***

<b>D</b>	<b>Bedienungsanleitung</b> Ringschleifen-Empfänger	Seite 2
<b>GB</b>	<b>Operating Instructions</b> Induction Loop Receiver	Page 6
<b>F</b>	<b>Mode d'emploi</b> Récepteur de boucle magnétique	Page 10
<b>NL</b>	<b>Gebbruksaanwijzing</b> Ringleidingontvanger	Pagina 14
<b>I</b>	<b>Istruzioni per l'uso</b> Ricevitore a loop circolare	Pagina 18
<b>E</b>	<b>Instrucciones de servicio</b> Receptor en bucle	Página 22

LPU-1  
Ringschleifen-Empfänger

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
LPU-1  
Induction Loop Receiver

**Special accessories:**

- A-4275-0 Induction loop receiver with 1 battery (without charging station)
- A-4973-0 Single charger for receiver
- A-4974-0 Multiple charger for 12 accus
- A-4955-0 Plastic shell for 5 stethoscope receivers

2

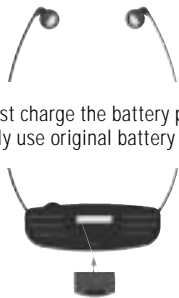
When you first charge the battery packs let them charge for at least 14 hours. Only use original battery packs.

3

Make sure that the stethoscope receiver is worn correctly


4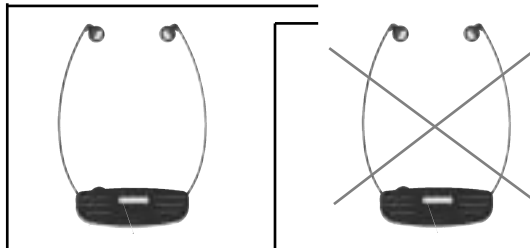


**Reception range:**

The induction loop receiver can receive signals which are transmitted by induction loop systems. This corresponds to a hearing aid in the »T« position. Ensure that the induction loop receiver is operated within an installed induction loop. The induction loop receiver does not work outside this range.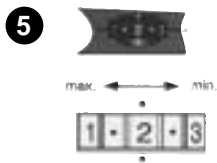

**Volume adjustment of the receiver.**  
**There is no ON/OFF switch.**

The receiver automatically switches on when worn.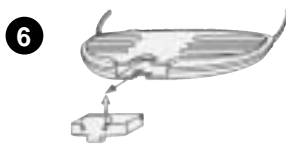

If the sound quality of the receiver becomes noisy and distorted the battery pack should be exchanged by a loaded one.


**Warning: do not force the battery pack into the charger or receiver. It fits easily when correctly inserted.**

**7** **Cleaning the hygienic covers**

The hygienic covers contained in the standard delivery package should be cleaned when they become dirty.

To do this use the Hygienic-SET offered as a special accessory (order no. A-4982-0).

**8** Please note the following information about your LPU-1:

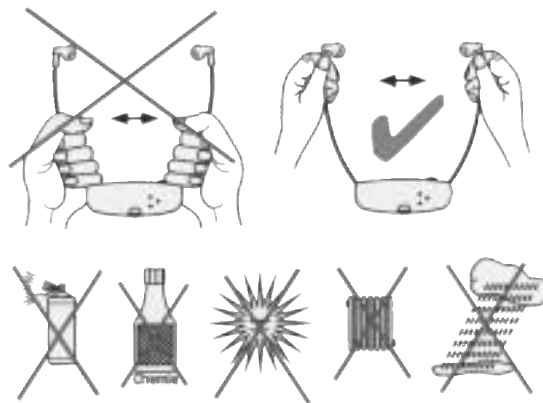If the system needs cleaning ONLY use a soft damp cloth.  
If the foam ear pieces become damaged they should be replaced.

**9** Technical data:

Frequency response	70 - 5400 Hz
Distortion	< 1 %
Noise level	typ. 60 dB
Battery charging time	approx. 14 h
Operating time	approx. 6 h
Maximum volume SPL	approx. 120 dBA
Weight	47 g

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**Humantechnik GHL AG**
ELECTRONIQUE S.A.R.L**