HUMANTECHNIK

Pocketalker® Ultra

D

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Seite 2

Kommunikationssystem

GB

Operating Instructions

Page 6

Communication System

F

Mode d'emploi

Page 10

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NL

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Pagina 14

Communicatiesysteem

I

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Pagina 18

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
E

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
Página 22

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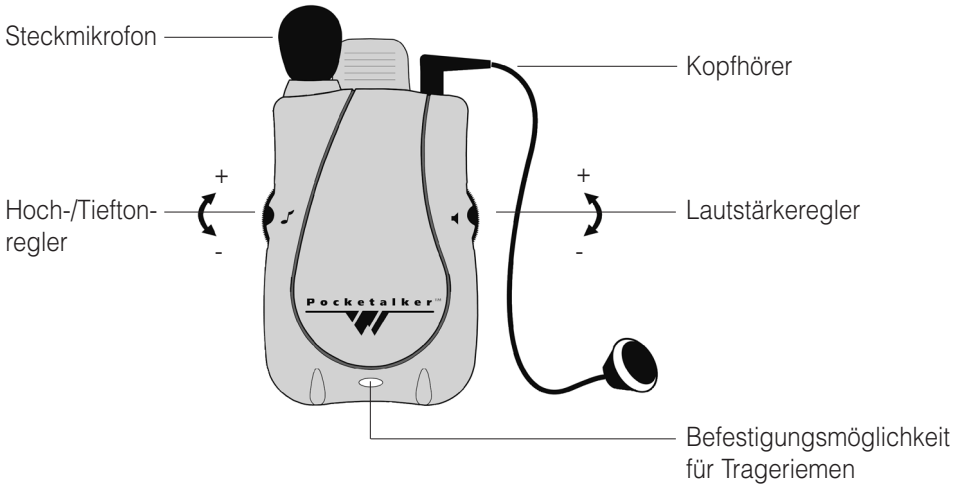
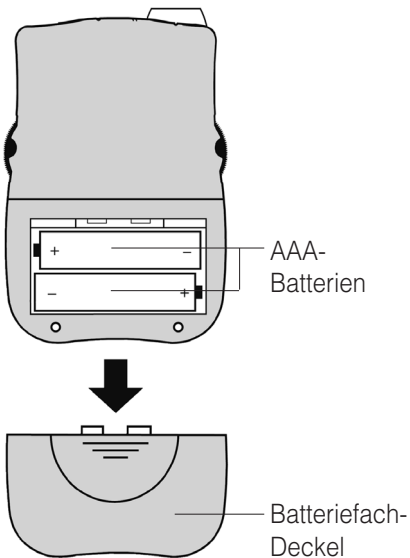
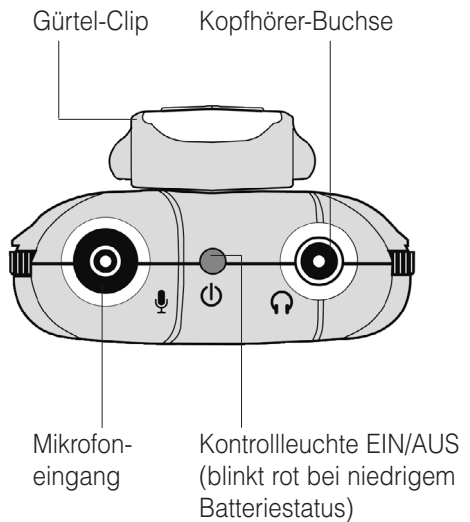
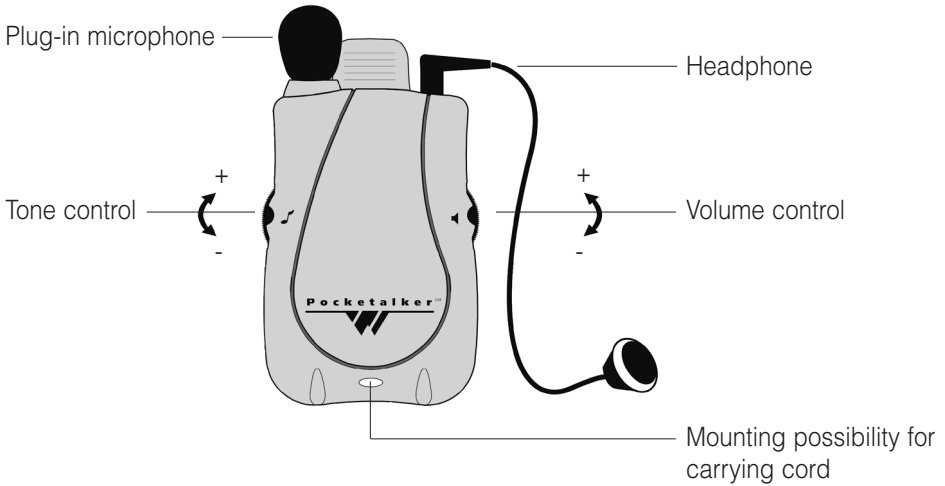
Operating controls - Back

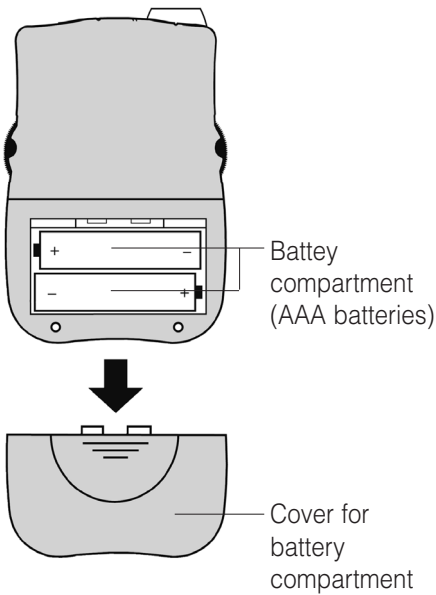Operating controls - Top

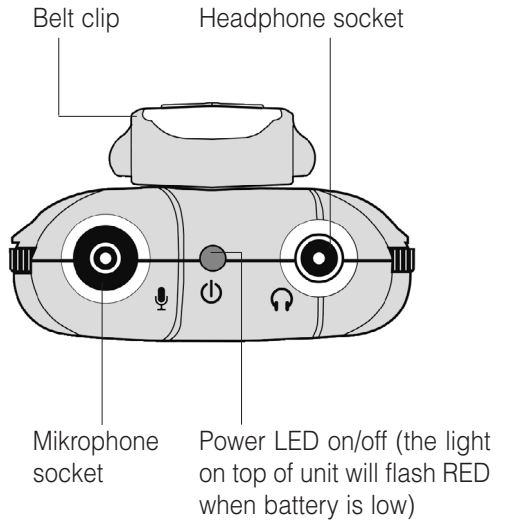Congratulations on purchasing your *Pocketalker® Ultra*. You have chosen a modern and reliable system. Please read through the operating instructions carefully to be able to start the unit correctly and to familiarise yourself with all of the systems features.

Standard components

Please check if all following components are included:

- Pocketalker® Ultra
- 2 x 1.5 V AAA batteries
- Plug-in microphone
- Headphone (Mono)
- Carrying cord
- Belt clip
- Operating instructions
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

Functional principle

The communication system *Pocketalker® Ultra* is meant to facilitate the participation of people with hearing loss, at conversations under difficult terms. Use the *Pocketalker® Ultra* to improve listening on a one-to-one basis, in small group situations, while listening to your favorite TV or radio programs, or while riding the car.

The sonic level of the system can be very high. Make sure the volume is turned down before the batteries will be put in or the headphones/microphone will be plugged or unplugged.

Inserting batteries

The amplifier is operated with two micro batteries (Typ AAA).

1. Open the battery compartment door:
Using your thumb, apply firm pressure on the ridges of the door and slide it down and away from the unit.
2. Insert the two batteries according to the polarity marks.
3. Close the battery compartment.

Please take care that the batteries will be inserted in accordance to the polarity marks. The correct placement of (+) and (-) terminals is displayed graphically inside the battery compartment.


Operating


Figure: volume control on/off

1. Plug the microphone and headset into the corresponding jack.
2. Turn on the *Pocketalker® Ultra* by turning the volume control counterclockwise. The red `on` indicator light on the top of the unit should be lit.
3. Turn up the volume until you are able to understand your dialogue partner best

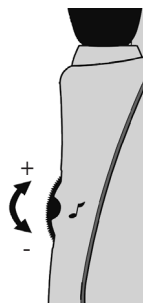Figure: tone control

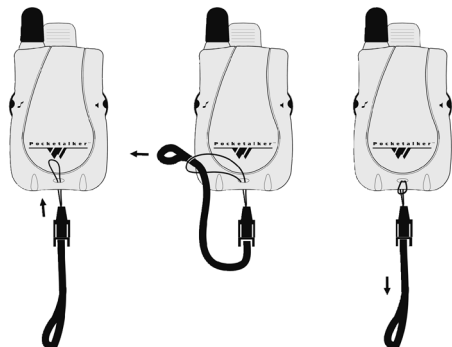4. Adjust the volume up or down according to your preferences. Adjust the tone according to your preferences by turning the tone control towards the corresponding direction.

The carrying cord enables you to carry the *Pocke-talker® Ultra* safely while keeping your hands free.

Hints and tips

The *Pocke-talker® Ultra* has been designed to be as easy to use as possible. However, if you have trouble, check the following:

1. Make sure the microphone is plugged into the 'MIC' jack and the earphone or headphone is plugged into the headphone socket
2. Make sure the batteries are not depleted and have been inserted correctly into the battery compartment. If rechargeable, make sure the batteries are sufficiently charged.

3. If you experience a howling or squealing sound, this is called feedback. To eliminate feedback, turn down the volume control or move the microphone further away from the earphone or headphone. Also, try turning the tone control down (or counterclockwise) to reduce the high frequency.
4. The headphone-socket is adapted for the mono-headphone, which is included in delivery. If you use stereo earphones or headphones, only one side of the headphones will work.

Maintenance and care

The *Pocke-talker® Ultra* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the *Pocke-talker® Ultra* where it will be exposed to full sunlight for long periods.

In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

Warranty

The *Pocketalker® Ultra* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Technical data

Dimensions (HxWxD): 86x57x22 mm

Colour: silver/grey

Weight: 71 g incl. batteries and plug-in microphone

Reserve energy: 100 h (the light on top of unit will flash RED when battery is low)

Amplification: max. 120 dB

Power supply: 2x 1,5 V mignon batteries (Type AAA)

This product complies with European Union norms.

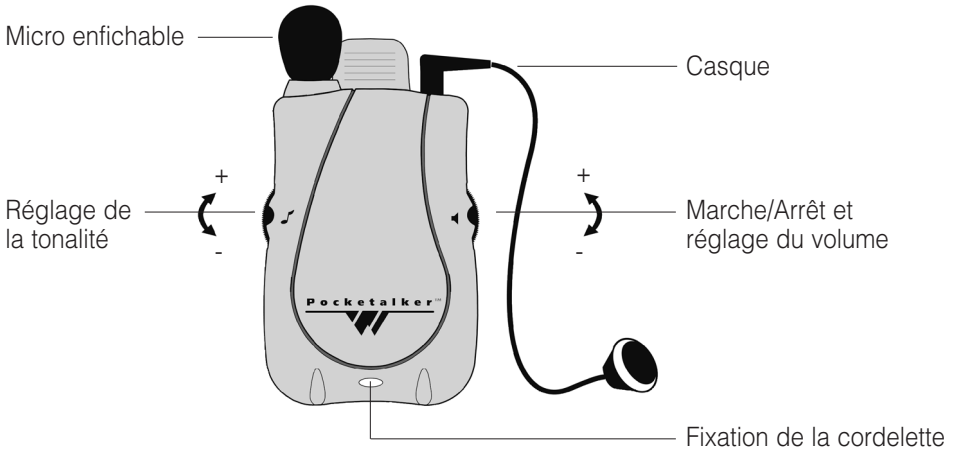
You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

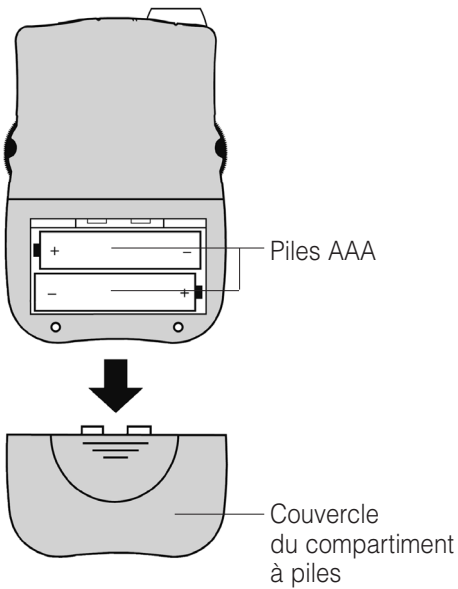
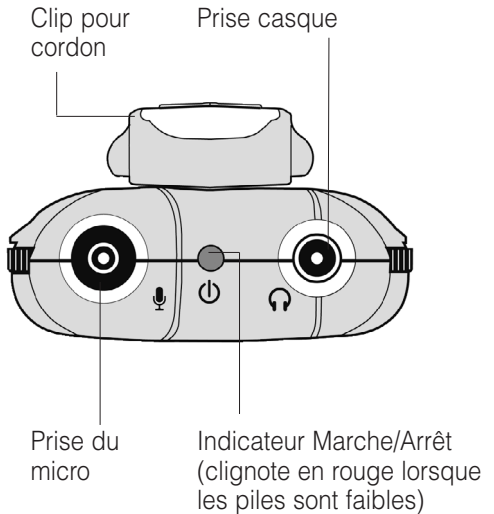
Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

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Humantechnik GHL AG