HUMANTECHNIK

soundshuttle

D

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Seite 2

Tragbares Ringschleifensystem

GB

Operating Instructions

Page 6

Portable induction loop system

F

Mode d'emploi

Page 10

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
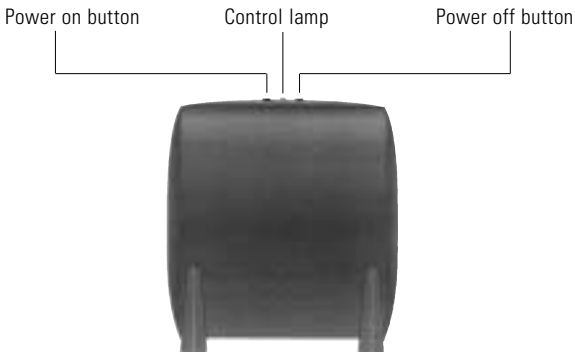
Operating elements - front

Operating elements - back

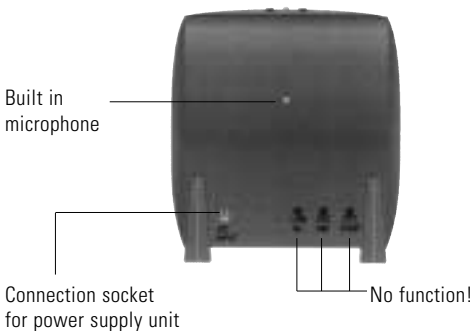We would like to congratulate you on purchasing your *soundshuttle*. You have chosen a modern, reliable device. Please first read these operating instructions carefully to initially start the device and to become familiar with all the system options.

How it works

The *soundshuttle* uses its built-in microphone to record sounds and emits them via the built-in induction loop. The signals emitted in this way can be received by a hearing aid that is set to the "T" or "MT" position.

Package contents

The following parts are included in the package:

- *soundshuttle*
- Power supply unit
- Operating instructions
- Guarantee card

Setting up the *soundshuttle*

The induction loop system is already completely installed and set up in the housing. All you have to do is to place the *soundshuttle* at the desired position between you and the person you are talking to and connect the power supply unit. Please note that the microphone on the back of the device should be facing the speaker.

Switching the device on

To switch the induction loop system on, press the green button on the top of the device. Once the device is switched on, the control lamp will light up green.

Recording / emitting signals

If sounds are received by the microphone on the back of the device and emitted by the device as magnetic signals, the control lamp will begin to flash orange.

Switching the device off

To switch the induction loop system off, press and hold the red button on the top of the device until the control lamp goes off.

Power supply for the *soundshuttle*:

The *soundshuttle* can be operated using the built-in rechargeable battery or the power supply unit. If the battery is fully charged, the device can be used for up to 4 hours. A dead battery has to be recharged for approximately 16 hours until it is fully charged once again. You can also use the *soundshuttle* while it is being charged, but please note that this will lengthen the time required to fully recharge the battery. The device can remain connected to the power supply voltage for as long as desired.

Maintenance and care

The *soundshuttle* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

Guarantee

The *soundshuttle* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer.

The warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorized by Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer as well as a copy of the sales slip. Always specify the product number in any event.

CE Marking

This device complies with the requirements of EU directives: 73/23/EEC, 89/336/EEC and 1999/5/EC. Compliance with these directives is indicated by the CE Marking on the device.

Disposal

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).


The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Specifications

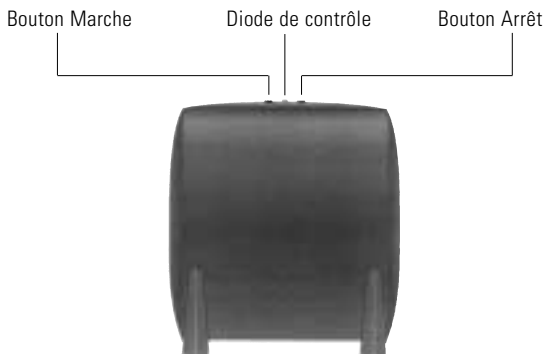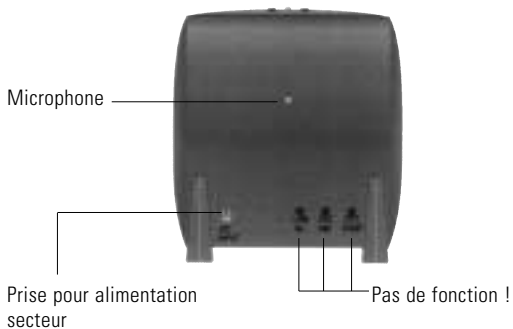
Dimensions:	180 x 170 x 65 mm (HxWxD)
Weight:	420 g
Microphone sensitivity:	Up to 60 dB \pm 3 dB
Power supply: Power supply unit:	Primary 100-240 V 50-60 Hz AC / secondary 15 V DC or built-in 12 V NiMH rechargeable battery
Output power:	max. 7 W

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Humantechnik GHL AG